

OsseoConduct™

Beta Tricalcium Phosphate Granules and Cortical Plates

OsseoConduct™ is a synthetic, porous, biocompatible beta tricalcium phosphate bone grafting material. OsseoConduct™ is resorbable and is intended for use as a general bone graft material. OsseoConduct™ is available in two forms, in a granular form and in a plate form. OsseoConduct™ granules and plates are designed to maintain shape and volume when used for ridge augmentation or socket grafting but are not appropriate for immediate support of removable dental appliances.

DIRECTIONS FOR USE:

- 1) Open the pouch and remove a packet containing OsseoConduct™. Reseal the pouch to maintain sterility of the remaining packets. The packets containing OsseoConduct™ are sterile on both inside and out and can be placed on the sterile operating surface
- 2) With sterile scissors open the packet containing OsseoConduct™ and place it in an appropriate sterile container
- 3) To facilitate handling, OsseoConduct™ can be wetted with sterile water, autogenous blood, bone marrow or a wetting agent of your choice
- 4) OsseoConduct™ in plate form is trimmed to form the desired ridge to maintain space permitting osteoconduction resulting in bone formation
- 5) OsseoConduct™ granules and plates maintain graft site volume while osseointegration occurs

PACKAGING:

OsseoConduct™ is packaged in sterile single use packets. The packets are contained in a sterile re-sealable pouch. The graft packets are sterile on both the inside and outside and can be placed in the sterile surgical field.

INDICATIONS:

OsseoConduct™ is intended for use as a maxillo-facial bone graft material in bone voids and for appositional bone growth.

CONTRAINDICATIONS:

OsseoConduct™ is not indicated where loading of the graft site is planned such as oral sites that are intended to be loaded immediately with dental prosthesis.

OsseoConduct™ has not been studied in pediatric patients.

WARNINGS:

OsseoConduct™ has not been studied in pediatric patients.

OsseoConduct™ is single use only, do not re-sterilize or reuse.

Do not use OsseoConduct™ if the package has been opened or damaged or has exceeded the expiration date.

Caution: Federal law restricts this device to sale by or on the order of a dentist or a physician.

Contact Information

STEINERBIO®

1051 Olsen St., Bldg. 3611 Henderson, NV 89011

866.317.1348 www.steinerbio.com

contactus@steinerbio.com