

Ridge Graft Kit™

Ridge Graft Kit™ Is Intended to be mixed with OsseoConduct™ bTCP standard size granules in a 1:1 ratio for ridge augmentations.

Each syringe of Ridge Graft™ contains 1cc of Graft material. Each packet of OsseoConduct™ bTCP contains 1cc of standard granules size 500-1000um.

Mixing Ratio: mix 1 syringe of Ridge Graft™ with 1cc of OsseoConduct™ bTCP granules.

DIRECTIONS

Either syringe can be used for dispensing graft material.



1. Do not compress powder syringe. Compressing the powder will not allow syringes to mix. Remove both caps from each syringe. Make sure powder in syringe is loose



2. Screw both syringes together. Push liquid into powder syringe and give the syringes a few vigorous shakes. Mix back and forth about 10 times until completely mixed



3.

4. After mixing, push entire graft material into either syringe and unscrew syringes
5. Dispense Ridge Graft into a sterile container and add the OsseoConduct bTCP granules and mix together
6. Apply mixed graft material to site immediately not allowing graft material to dry

Ridge Graft Kit™

Ridge Graft™ is a dual phase calcium phosphate based synthetic bone graft material. Ridge Graft™ is designed to quickly and safely treat and retain alveolar bone, stimulate bone formation and reduce post-operative complications. Ridge Graft™ is designed to be quickly and completely resorbed. Ridge Graft™ must be stored in the refrigerator.

INDICATIONS: Ridge Graft™ is a bone grafting material intended for ridge augmentation and must be mixed with Osseoconduct bTCP granules standard size.

CONTRAINDICATIONS: Ridge Graft™ is contraindicated in individuals who are unable to follow post op instructions.

PRECAUTIONS: Do not overfill defects. Do not leave defect open. Do not compromise blood supply to the defect area. Ridge Graft™ has not been studied in pediatric patients Ridge Graft™ is not intended for immediate load-bearing.

WARNINGS: Ridge Graft™ is single use only. Do not re-sterilize or reuse. Do not use Ridge Graft™ if the package has been opened or damaged or has exceeded the expiration date. Do not use if the temperature indicator on the packaging has turned black, indicating possible damage to the regenerative potential of Ridge Graft™

It is not advised to mix Ridge Graft™ Kit with other graft materials other than Steiner Biotechnology bone graft materials.

Federal law restricts this device to sale by or on the order of a dentist.

If the gray dot on the product box turns black, the efficacy of the graft material may have been compromised. Please discard and notify us immediately.

CONTACT INFORMATION:

STEINERBIO®

1051 Olsen St., Bldg. #3611
Henderson, NV 89011
Ph 866.317.1348

email: contactus@steinerbio.com
www.steinerbio.com